



# EASY PULSE®

Efficient and compact mechanical CPR  
thanks to technological innovation

*Latest Generation of  
Automatic CPR*




**SCHILLER**  
The Art of Diagnostics



# EASY PULSE®

SCHILLER has taken the next step in improving CPR quality further with the launch of the unbelievably small and light mechanical chest compression device: the EASY PULSE.

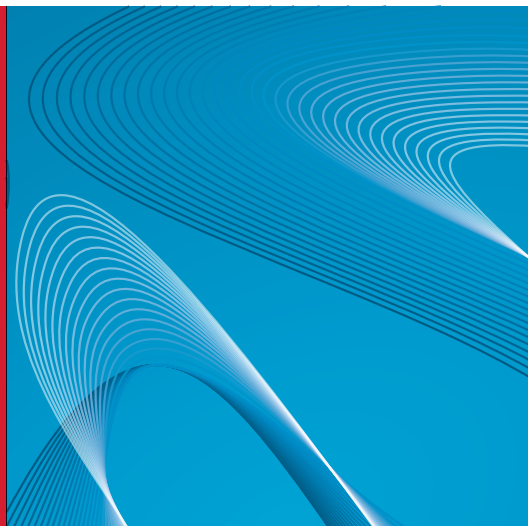

Performing manual chest compressions well for an extended period of time is almost impossible. Not only is it physically demanding, but other actions are also required simultaneously, such as vital signs monitoring and defibrillation.

SCHILLER's EASY PULSE is the solution for more effective resuscitation: this portable, stand-alone device delivers chest compressions automatically at a consistent rate and depth. It is directly attached to the patient's upper body and can thus be used in any situation. Especially during transport, a high-quality uninterrupted chest compression can be guaranteed.

The EASY PULSE is such a simple device that a single person can operate it. Its size and weight are unparalleled: the EASY PULSE weighs 3.5 kg, opening up entirely new possibilities e.g. for air rescue services.

## LATEST GENERATION OF AUTOMATIC CPR DEVICES


The EASY PULSE has a unique combination of stamp and band which allows a 3D compression and therefore maximum efficiency.

The weight of the device is evenly spread over the baseplate and not on a single point and therefore has no influence on chest compressions nor on the recoil.



## SLIDER

The slider enables a relieved preparation of the patient for mechanical chest compressions. It also takes very little time to attach the device thanks to the practical buckle system – less than 15 seconds for trained personnel.

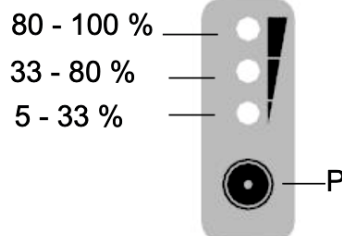

## BATTERY CHECK

This check can be performed at any time, even when the battery is not in use.



## CHARGER

Charger for 2 rechargeable batteries. Slot 1 for fast charging in approx. 45 minutes; if two batteries are charged simultaneously, normal charging duration is approx. 2 h. One full battery charge lasts for about 45 minutes operation.



## EASE OF USE

In an emergency, every move has to be right. The EASY PULSE is simple and intuitive to operate: pressing two buttons is enough to initiate resuscitation immediately.



## EXTERNAL POWER SUPPLY AND DIRECT CONNECTION CABLE

A power supply unit for 100VAC to 240VAC input voltage is supplied. The EASY PULSE can be directly operated using a 12VDC 48VDC power source via a special connection cable.

## TECHNOLOGICAL INNOVATIONS

### Lightweight

- ❖ Unbelievably small and light electrically driven CPR device, weighing 3.5 kg
- ❖ Ideal for limited spaces

### Comprehensive

- ❖ Multidirectional chest compressions (3D)
- ❖ High-quality chest compressions of consistent rate and depth, which is impossible to achieve with manual compressions
- ❖ Possibility of 30:2 compression-ventilation cycle

### Simple

- ❖ Thanks to slider and buckle system, it can be easily attached to the upper body
- ❖ Easy to position easy to operate

### Additional features

- ❖ Replaceable battery with charge level indicator
- ❖ Autonomous operation for 45 minutes (with one battery)
- ❖ Connections: external DC input, USB





SCHILLER, founded in 1974 by Alfred E. Schiller is a successful group with 30 subsidiaries and a global sales network. Today, SCHILLER is a world-leading manufacturer and supplier of devices for cardiopulmonary diagnostics, defibrillation and patient monitoring as well as software solutions for the medical industry.

Over the last 27 years, SCHILLER has established itself as a leader in the medical technology sector in India. With 400+ employees, two production centres and a R&D facility SCHILLER makes state-of-the-art healthcare equipment accessible through a network of 100 sales & service personnel spread across more than 45 locations. Our product range covers Critical Care, Anaesthesia, Emergency Care, Cardiology, Pulmonology, Radiology and Robotics.

Scan here  
to know more



Part No. B4.100005  9/2024 Ver. 1

All registered trademarks acknowledged.

**SCHILLER**  
The Art of Diagnostics

Swiss H.Q.: SCHILLER AG, Altgasse 68, P. O. Box 1052, CH -6341 Baar, Switzerland

Indian H.O.: Schiller Healthcare India Pvt. Ltd., Advance House, Makwana Road, Andheri (East), Mumbai-59. Tel.: + 91 - 22 - 61523333

Factory: No. 15/5 & 15/6, Vazhuthavur Road, Kurumbapet, Puducherry - 605 009, India

Regional Offices: Mumbai: 022-61523333 Email: sales.west@schillerindia.com Delhi: 011-41062067 / 09312432205 Email: sales.north@schillerindia.com Kolkata: 033-23593102 / 033-23593103 Email: sales.east@schillerindia.com Chennai: 044-28232648 / 044-28311021 Email: sales.south@schillerindia.com Ahmedabad: 079- 35337949 / 079- 35337950 Email: sales.west@schillerindia.com Puducherry: 0413- 220 2680 / 0413- 229 2940 Email: sales.south@schillerindia.com

CIN : U33110MH1997PTC111307

 sales@schillerindia.com, help@schillerindia.com  www.schillerindia.com  1800 209 8998

Follow us on :  /SchillerHealthcareIndia  /schiller\_healthcare\_india  /schillerhealthcareindia  /schiller\_healthcare\_india  @schillerindia1074